4a medical®  
producing health for the world

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# ABOUT US

*Founded in 1997, Dört-A Tıp Malzemeleri Sanayi İth. İhr. Tic. İtd. Şti., started its commercial life by producing Sterilization Reels with the brand name of 4A Medical. It has diversified its product range with high quality products such as Sterilization Pouches, Tyvek® Rolls, Sterilization Control Products and Drain Systems in order to meet the needs of healthcare industry. Its products has been approved by Turkish and the World medical authorities.*

*The products manufactured by 4A Medical, are composed of the best raw materials, high engineering and productive workmanship. In the light of science, 4A Medical collaborates with leading universities in Turkey in order to develop new products. Its production is regularly checked by the quality department and independent auditing institutions. Hence 4A Medical is able to satisfy its clients with the quality of state of the art products.*

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
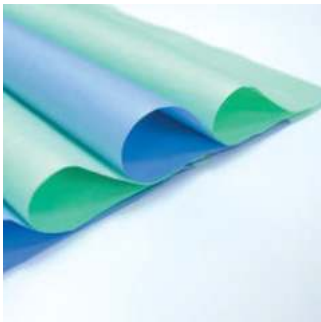
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
*4A Medical is a reliable company providing medical consumables to its partners operating in more than 50 countries. Caring to build global collaborations, 4A Medical attends the most popular international fairs such as MEDICA in Düsseldorf and Arab Health in Dubai. Additionally 4A Medical is a member of the Sterile Barrier Association (SBA) which is the European trade organization for companies who produce Sterile Barrier Systems. Hence 4A Medical is well known as the world leading manufacturer of sterilization packaging and control products.*

**4a medical**<sup>®</sup>  
producing health for the world

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*4A Sterilization Reels provide sterile storage in packages and keep package sterile for 6 months and do not permit microorganisms to enter inside the package after sterilization cycle.*

*The process indicators that are printed on the side of the Sterilization Reels are water based and conforms with ISO 11140.*

*4A Sterilization Reels are used for steam, ethylene oxide, formaldehyde sterilization methods. The sterilization conditions should be determined by the end user regarding to material to be sterilized.*

*4A Sterilization Reels meet ISO 11607 standards and they are manufactured using state-of-art technology with the highest quality.*

*Products can be produced with your trademark*

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*4A Sterilization Flat Reels can be produced from medical paper as 60gr/m<sup>2</sup> veya 70gr /m<sup>2</sup> and 52 micron, PP+PET green film. Any dimension between 5cm - 60cm can be made on request. On request, 100 m length can be produced as well.*

**4AFR**

**4a medical**®  
producing health for the world

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*4A Sterilization Gusset Reels can be produced from medical paper as 60gr/m<sup>2</sup> or 70gr /m<sup>2</sup> and 52 micron, PP+PET green film. Any dimension between 7,5cm - 50cm can be made on request. Gusset Reels has increased filling volume and can hold an increased amount of volume in comparison with Flat Reels.*

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*4A Sterilization Flat Pouches can be produced from medical paper as 60gr/m<sup>2</sup> or 70gr /m<sup>2</sup> and 52 micron, PP+PET green film. 4A Sterilization Pouches are customized with regards to their width and length and they are ready-to-use.*

*4A products comply with ISO 11607 standards while process indicators meet ISO 11140 standards.*

*Products can be produced with your trademark.*

**4a medical**<sup>®</sup>  
producing health for the world

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*4A Sterilization Flat Pouches are products in which three sides are closed and one side needs to be closed by final user using closing machine before sterilization process. 4A Sterilization Flat Pouches are cut and packed as 100 pieces wrapped in film according to requested dimension.*

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*4A Sterilization Gusset Pouches are products in which three sides are closed and one side needs to be closed by final user using closing machine before sterilization process. Gusset Pouches are suitable for materials with increased filling volume and they are cut and packed as 100 pieces wrapped in film according to requested dimension.*

**4AGP**

**4a medical**<sup>®</sup>  
producing health for the world

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*4A Sterilization Self Sealing Pouches are the products which can be sealed with their self-adhesive feature on the closing part of pouch without need of sealer.*

*4A Sterilization Self Sealing Pouches are cut and packed as 100 pieces wrapped in film according to requested dimension.*

**4ASSP**

**4a medical**®  
producing health for the world

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*4A Sterilization Reinforced Reels has more resistance than normal reels and they are produced from 60g non-woven paper and 52 micron film. This kind of reels offer a high tearing resistance and high permeability.*

*4A Sterilization Reinforced Reels are manufactured with one side non-woven and one side film. It is considered a single use product and can be used in sterilization process with steam, ethylene oxide and formaldehyde. Indicators for each process are printed on reels.*

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### ***Sterilization Self Sealing Reinforced Pouches***

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*4A Sterilization Self Sealing Reinforced Pouches are the products which can be sealed with their self-adhesive feature without need of sealer. With high resistance and high permeability it is specially produced for heavy and bulky product.*

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*4A Sterilization Reinforced Pouches are products in which three sides are closed and one side needs to be closed by final user using closing machine before sterilization process. With high resistance and high permeability it is specially produced for heavy and bulky product. 4A Sterilization Reinforced Pouches are cut and packed according to requested dimension.*

**4ASSRP**

**4a medical®**  
producing health for the world

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*4A Tyvek Reels are high quality products manufactured from tyvek paper and 52 micron white (transparent) (PET+PE) film to be used in plasma sterilization.*

*While 4A Tyvek Reels are suitable for low temperature gas plasma sterilization, they can be also used for ethylene oxide and gamma sterilization if needed.*

*Any dimension between 5cm - 50cm can be made on request. 100m length can be produced as well on request.*

*Products can be produced with your trademark.*

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*4A Tyvek Pouches are products in which three sides are closed and one side needs to be closed by final user using closing machine before sterilization process. 4A Tyvek Pouches are cut and packed according to requested dimension.*

**4ATP**

**4a medical®**  
producing health for the world

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*4A Crepe Paper is product that can be used for wrapping materials to be sterilized in steam, ethylene oxide and formaldehyde.*

*Our products which are manufactured with different length and 60gr on the demands of users, provide significant convenience for the users in the sterilization units.*

*4A Crepe Papers can be produced in blue and green colors.*

*Products can be produced with your trademark.*

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*SMS Wrapping Papers are packing covers used in the sterilization with plasma. It is available as 43 gr.*

*They can be manufactured with different length according to demand of the user. Only blue color is available.*

*Products can be produced with your trademark.*

**4ASMS**

**4a medical®**  
producing health for the world

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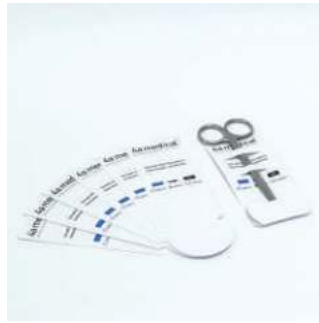
*4A Sterilization Control Products are used to supervise sterilisation process.*

*4A Sterilization Control products are produced with water based non toxic paint. Our indicators that conform with ISO 11140-1 certificates are products that provide convenience to users to evaluate the sterilization process and result.*

*All products can be produced with your trademark.*

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*4A Type 4 Steam Chemical Indicators are the indicators used inside sterilization packages during the steam sterilization and when multi parameters are obtained, the color turns into its reference color. 4A Type 4 Steam Chemical Indicators allow the end user to identify incorrect packaging, loading problems and sterilization failures.*

*Besides standard Type 4 Steam Chemical Indicator we also have Self Adhesive Type 4 Steam Chemical Indicator with self adhesive feature on the back for easier documentation as well as Perforated Type 4 Steam Chemical Indicator*

**4a medical**<sup>®</sup>  
producing health for the world

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*4A Type 5 Chemical Integrators are the indicators which give proximate result to the biological indicator and they operate in direct proportion to biological death.*

*This product that is used in a package during steam sterilization process consists of the chemical mixture on the part of the acceptance area if the sterilization is successful and on the part of reject area if the sterilization fails.*

*There is no reference color in integrators, and the results can be interpreted according to the mixture migrating through the window.*

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
*4A Type 5 Chemical Indicators are cycle verification indicators that confirm the presence or absence of specific time and temperature parameters during a cycle. The products allow the end user to identify incorrect packaging, load problems and sterilization failures. 4A Type 5 Chemical Indicators are designed to operate at steam cycle 121°C and 134°C operation conditions.*

*Besides standard Type 5 Steam Chemical Indicator we also have Self Adhesive Type 5 Steam Chemical Indicator with self adhesive feature on the back for easier documentation.*

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4A Type 6 Steam Chemical Indicators are the products that are put into each sterilization package and that indicate success of the sterilization. 4A Type 6 Steam Chemical Indicators are cycle verification indicators that confirm the presence or absence of specific time, temperature and pressure parameters during a cycle.

While Type 6 Steam Chemical Indicator 3,5 minutes operates up to 15 minutes at 121°C, up to 3,5 minutes at 134°C, Type 6 Steam Chemical Indicator 7 minutes operates up to 20

minutes at 121°C, up to 7 minutes at 134°C.

We have the following 4A Type 6 Steam Chemical Indicator types:

Besides standart Type 6 Steam Chemical Indicator 3,5 minutes we have Self - Adhesive Type 6 Steam Chemical Indicator 3,5 minutes and Perforated Type 6 Steam Chemical Indicator 3,5 minutes.

Besides standart Type 6 Steam Chemical Indicator 7 minutes we have Self - Adhesive Type 6 Steam Chemical Indicator 7 minutes and Perforated Type 6 Steam Chemical Indicator 7 minutes.

**4a medical**<sup>®</sup>  
producing health for the world

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*4A Type 6 Registration Cards are the products that are put into each sterilization package and that indicate success of the sterilization. 4A Type 6 Registration Cards are cycle verification indicators that confirm the presence or absence of specific time, temperature and pressure parameters during a cycle.*

*Type 6 Steam Chemical Indicator Registration Card 3,5 minutes operates up to 15 minutes at 121°C, up to 3,5 minutes at 134°C. It has additional space for adding notes. Thanks to its self adhesive property it is easy for documenting sterilization results.*

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*4A Bowie Dick Test Packs are the testing packages that should be used before starting daily machine sterilization process for steam sterilization devices. Bowie Dick Test Pack is used for the purpose of testing whether there is air leakage and efficient steam penetration in the steam sterilizers.*

*4A Bowie Dick Test Packs color change allows assurance of whether the steam sterilizers are ready for the sterilization or not.*

*Different kinds of Bowie Dick Test Packs are available in 4A production. These are 4A Bowie Dick Test Pack 200 (wrapped in medical paper, wrapped in crepe paper and BD in box) and 4A Bowie Dick Test Pack 400 (wrapped in medical paper and wrapped in crepe paper) and 4A Bowie Dick Test Pack EWS (Early Warning Test Sheet)*

Note: Products can be produced with your trademark

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
**4APCD / 4AHLX**

4A PCD- Helix Load Control Test has a self adhesive indicator on the far end. The PCD test kit has a 1 m long hose, and the Helix test kit has a 1.5 m long hose. There are 100 indicators in each package.

In this product used for load control, the adhesive indicator is placed at the far end of the test apparatus, steam inlet only can be at the beginning of the hose and it can reach the chemical indicator at the end of the hose by passing at a single point.

If the chemical indicator color change is achieved despite the difficult conditions, it is an important reference for the sterilization process in that load and should be recorded as a record for the successful sterilization of the load.

Types of 4A PCD - Helix Load Control Tests are as follows:

PCD - Helix Load Control Test with Bowie Dick Indicator

PCD - Helix Load Control Test with Type 6 Indicator

PCD - Helix Load Control Test with Ethylene Oxide Indicator

PCD - Helix Load Control Test with Plasma Indicator

**4a medical®**  
producing health for the world

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4ABI

Biological indicators are test systems containing viable microorganisms providing a defined resistance to a specific sterilization process.

A biological indicator provides information on whether necessary conditions were met to kill a specified number of microorganisms for a given sterilization process, providing a level of confidence in the process.

Biological indicators, depending on the specific type, can be used for various sterilization processes using steam, hydrogen peroxide gas or

ethylene oxide.

We have the following 4A Biological Indicator types:

4A Steam Biological Indicator (3h) is specifically designed for the monitoring steam sterilization processes at 121 – 134 °C. The final readout of a processed biological indicator with respect to the indicator type should be made after 3 hours (4ARSBI) or 24h (4ASBI) of incubation at 60+/- 2 °C.

4A Ethylene Oxide Biological Indicator (4AEOBI) is specifically designed for the monitoring of ethylene oxide (EO) sterilization processes. The final readout should be made after 48 hours of incubation at 37+/- 2 °C.

4A Plasma Biological Indicator (4APBI) is specifically designed for the monitoring of Hydrogen Peroxide sterilization processes. The final readout should be made after 24hours of incubation at 60+/- 2 °C.

**4a medical**®  
producing health for the world

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*4A Ethylene Oxide Indicators which are Type 5 indicators are chemical indicators which gives information about success of sterilization by using inside sterilization pouches together with ethylene oxide.*

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*4A Formaldehyde Indicators are indicators used inside sterilization pouch in formaldehyde sterilization.*

*These indicators which are included in Type 4 according to ISO 11140-1 standards give information to user whether material inside the package is sterile or not.*

**4AFOI**

**4a medical**<sup>®</sup>  
producing health for the world

## Hidrojen Peroksit (Plazma) Kimyasal İndikatörü *Hydrogen Peroxide (Plasma) Chemical Indicators*

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*4A Plasma Indicators which are Type 4 indicators are chemical indicators which are used in sterilization inside package for sterilization with hydrogen peroxide.*

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*4A Steam Documentation Labels are labels with double adhesive support and process indicator for steam sterilization. Each roll contains 500 pieces of double self adhesive stickers. The indicator printed on the sticker is indicator Type 1.*

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*4A EO Documentation Label is designed as double self-adhesive label printed with indicator ink and used for documentation in Ethylene Oxide sterilization. Each roll contains 500 pieces of double self adhesive stickers.*

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*4A Plasma Documentation Labels are labels with double adhesive support and process indicator for plasma sterilization. Each roll contains 400 pieces of double self adhesive stickers.*

**4a medical**<sup>®</sup>  
producing health for the world

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*4A Documentation Label Gun with 3 line is for use with the 4A Documentation Labels. 4A Documentation Label Gun provides an easy handling and printing solution for indicator labels used for recording purpose. The label gun can be used to print important traceability data on the label such as sterilization/expiry date, operator code and autoclave/cycle no on the label. The ink cartridges can be replaced.*

**4a medical**<sup>®</sup>  
producing health for the world

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4A Autoclave Tape is an adhesive tape used in autoclaving to indicate whether a specific temperature has been reached. 4A Autoclave Tapes has a Type 1 indicator.

4A Autoclave Tape works by changing color after exposure to temperatures commonly used in sterilization processes. Tape also features a chemical indicator to verify that it has gone through the sterilization process.

They are a safe solution for closing wrapping materials.

4A Autoclave Tape

assortments are as follows:

Steam Autoclave Tape: 19mm x 50m (4ASAT1950) and 25mm x 50m (4ASAT2550);

Ethylene Oxide Autoclave Tape: 19mm x 50m (4AEOT1950) and 25mm x 50m (4AEOT2550);

Formaldehyde Autoclave Tape: 19mm x 50m (4AFOT1950) and 25mm x 50m(4AFOT2550);

Hydrogen Peroxide (Plasma) Autoclave Tape: 19mm x 25m (4APT).

Autoclave Tape without indicators: 19mm x 50m (4AAT1950) and 25mm x 50m (4AAT2550);

\*Note: Optional sizes are available upon request.

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*Instrument Protectors protect sharp pointed surgical instruments from damage and reduces possibility of pouch puncture during sterilization.*

*It allows for viewing and easy identification of each instrument and keeps instruments open to permit steam penetration.*

*4A Instrument Protector assortments: 50mm x 125mm (4AIP50125) and 75mm x 150mm (4AIP75150)*

*Note: Products can be produced with your trademark*

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
*4A Pro Test which is developed by our company, is an indicator which can be used safely at hospitals to determine the effectiveness of the washing disinfection operations before sterilization. Protein residues on medical equipment are used by bacteria which results with an increase in the number of bacteria that causes a drawback for a proper sterilization. In addition, these kind of residues on these medical equipment are the sign of life forms that causes the risk for infection. To prevent these kind of infections, identification and cleaning of protein residues are obligatory. After washing with protest the question that residues on medical equipment are whether protein or not will be answered.*

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
*Before the sterilization process of medical equipment and surgical instruments, there is a prewashing procedure. By the means of this procedure, blood residues remaining on especially surgical instruments that cause cross infection is surely cleaned. After Washing/Disinfection steps, there might be some residues or contaminants left in varying colors that can be observed. It is very essential to identify if the residue is of blood or corrosion, in order to carry out sterilization properly or otherwise to dispose the equipment.*

*This blood residue test developed in our laboratories is an indicator which changes color only with a specific enzyme in the blood. By using this test, it is very easy to identify trace amounts of blood. Without a need of long incubation time, our test is able to identify up to-0.1micrograms of blood in residue, faster than any other test methods, that is as short as 10 seconds.*

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*Cleaning and disinfection are critical aspects of the instruments reprocessing loop. That is why 4A Washer Test is designed for validation and the routine monitoring of the cleaning efficiency of cleaning processes in washer disinfectors for surgical instruments.*

*4A Washer Test is for fast*

*and clear visual assessment of cleaning results. Each washer test is single use only.*

*4A offers 3 different types of washing tests:*

*4A Washer Test: This Washing Test does not have its own plastic frame, so it is necessary to use other frames or boxes. For this Wash Test 4A is also manufacturing 4A Wash Test Holder.*

*4A Washer Test with Glass: Washer Test with Glass has its own frame and does not need any other frame or box.*

*PLS Washer Test Indicator (4APLS): PLS Washer Test Indicator has been carefully designed to perform in an equivalent manner to 'test soils' by providing a realistic multiple challenge for the washer disinfectant machine. 4A is also producing PLS Washing Test Holder for 4A PLS Washing Test.*

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*Cannula Control tests the effectiveness of washing devices as simulations of cannulated control lumen instruments.*

*Thanks to the Cannula Control product, foresight (information) about the cleaning of the interior of lumen instruments is obtained. Washer Test Indicator is used to test the performance of the washing device. It is understood that the inner parts of lumen instruments are not cleaned if there is a blood stain on the test indicator.*

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*Sonicontrol is a product which evaluates the performance of Ultrasonic Washer Machines.*

*It gives instant results by changing from green to yellow. If there is no color change, it indicates that there is a problem in the washing device. The liquid inside the tubes shows that the machine is working successfully. The final control color change occurs with the energy given by the ultrasonic device.*

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
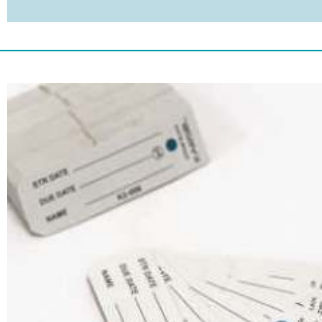
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
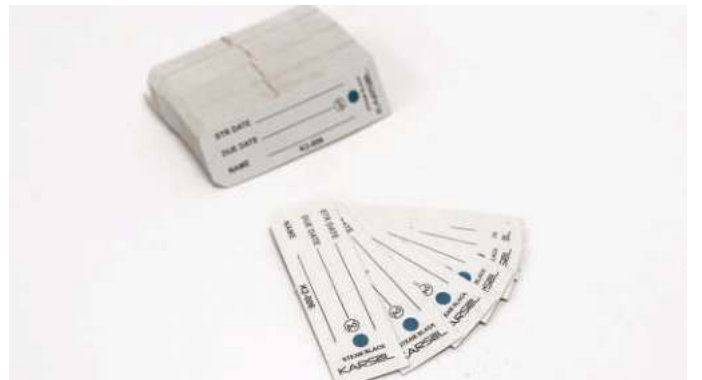
*4A Security Seals are used in order to secure that container is not opened by un-authorized staff. Container can be opened only by breaking the seal. Security seals are guaranteeing the sterility of materials from the moment of sterilization till the actual use.*

*4A is offering Security Seals with and without indicators.*

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*4A Container Labels are an important aid to label and mark the container. 4A Container Label with process indicator is used to place outside each container for an easy identification of the content. Details like sterilization date, sterilization expiry date, operator's name, lot number, autoclave details can be written down on the label. The chemical process indicator is giving the indication whether it has been sterilized or not.*

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4A Container Filters are soft and porous single use medical grade paper filters used with sterilization containers in clinics and hospitals for sterilization by autoclaving of instruments and other materials for surgical use. They are suitable with all types of containers and baskets.

4A Container Filters are placed in the lid and body of the sterilization container to allow steam penetration during sterilization and to maintain sterility post sterilization.

4A Medical provides circular shape container filters and rectangular shape container filters. Rectangular shape container filters have assortments of 120mm x 230mm and 165mm x 230mm.

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
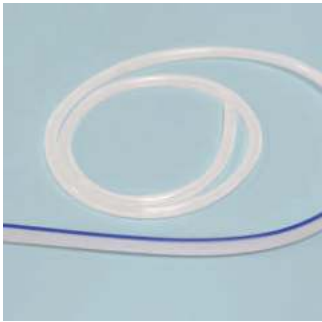
*The product, which has a collection capacity of 2000 ml has been produced as a mono block body to cancel out the possibility of air bubbles. The product has definitely been produced from a non breakable raw material and is positioned on a wide plate to prevent tilting over.*

*With the flexible hose and spiral property, it is easy to use. Air leakage controls of the products are made with special air leakage control tests developed by us. There is a hanger to make it easier to carry. There is a funnel for water filling.*

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
*It consists of a catch pit of 200cc, 400cc, 600cc or 800cc and a blower and it is used with 10, 12, 14 or 16F trocars.*

*It includes a single or double trocar optionally. It includes a connector that can be cut according to the drain size and an optional hanger can be attached to the drain.*

*After the special tests made, there is no deflation problem in the transferred products.*

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*Our aspiration set products are produced as with vacuum or without vacuum upon request.*

*Aspiration tips consist of four side holes, one part and hard structure.*

*Aspiration hose is 210 cm long and could be produced in requested sizes as well.*

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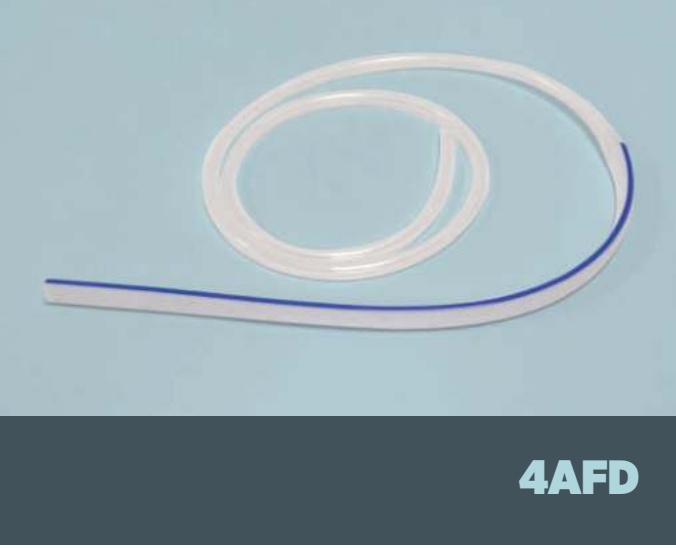
*Thorax drains are the products used in the thoracic surgery along with the chest bottle products in accordance. They could be produced in sizes 22F, 24F, 26F, 28f, 30F, 32F, 34F and 36F. There is X ray line on the product.*

*Depending on the request, they could be produced as PVC or silicon.*

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
*Made of 100% silicon material, flat drains are in the sizes of 5mm, 7mm, 10mm and 13mm and used with 125CC, 200CC, 250CC, 400CC PVC bomb reservoir upon request. There is X ray line on the product.*

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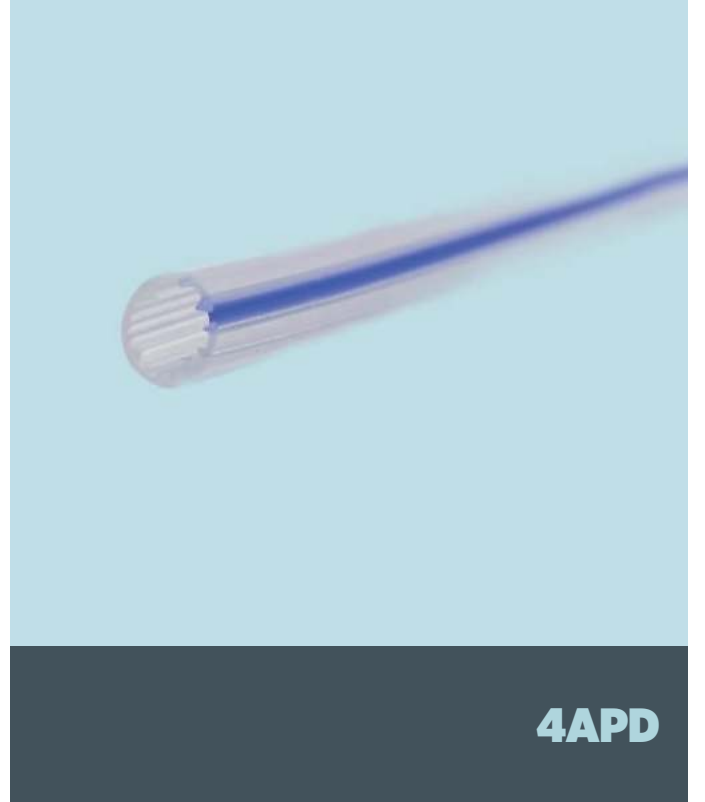*Silicone Penrose Drain is single piece. Silicone Penrose Drain is made from 100% silicone. Length of drain is 30 cm. Available sizes are 6 mm, 8 mm, 10 mm and 12 mm. There is X ray line on the product.*

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*T Drain is single piece. T Drain is made from %100 silicone. Lengths of drain are 30 cm and 50 cm. There are 9 dimensions and the dimensions of product are 8F, 10F, 12F, 14F, 16F, 18F, 20F, 22F and 24F. There is X ray line on the product.*

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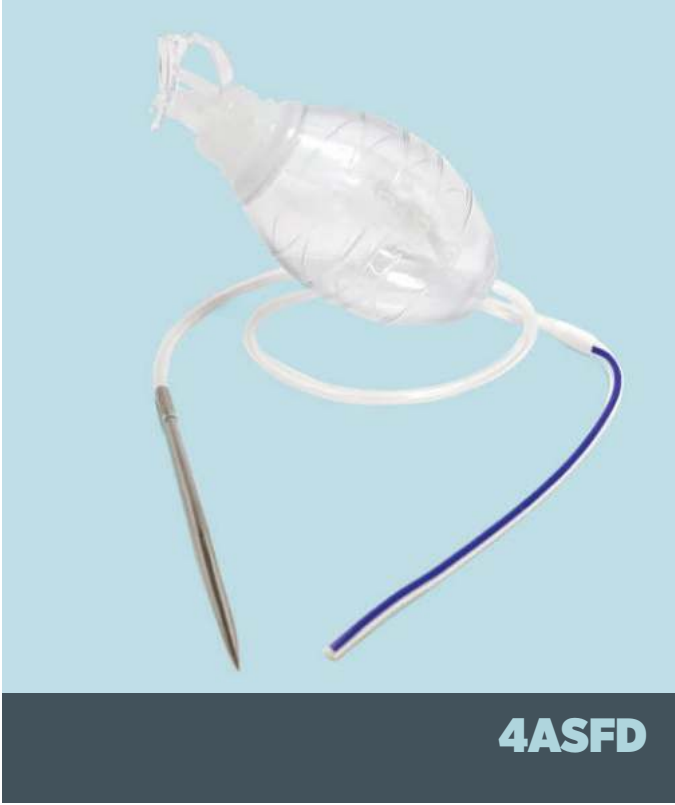
*PVC Bomb Reservoir has stopping valve. Bomb reservoir is single piece. Part of reservoir is made from Medical Grade PVC. Part of lid is made from Moblen. Part of pin is made from ABS. Available sizes are 125cc, 200cc, 250cc and 400cc.*

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1. *Made of medical grade silicone.*
2. *With X-Ray opaque line, used for closed wound suction draining.*
3. *These internal four channels, or flutes, are utilized to quickly remove exudates.*
4. *Hub of the tube minimizes pain associated with device removal.*
5. *With or Without Trocar.*
6. *400cc bomb reservoir.*
7. *Available sizes, 10 FR, 15 FR, 19 FR, 24 FR, 28 FR.*

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*4A Esu Pencil is for one time use and hand controlled. In surgical procedures yellow button is used to burn and blue button is used to cut coagulation process. With 3 inputs, it is compatible with all cautery devices.*

*4A Esu Pencil is used for cutting and fractionation living cell with sudden high frequency which is produced from electro devices in the surgery application.*

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## Straight Dren Torbalı / Torbasız Straight Drain With / Without Bag

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The straight drain product can be produced with or without bag in the dimensions of 15F,18F,21F,24F,27F,30F,32F,33F, 36F,40F in 50cm, 90cm and 100cm lengths. It is produced optionally as PVC or silicone. There is an X-ray line on the product.

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